



Precision Health Scale

UC-324NFC

Instruction Manual

Contents

Before Use

Precautions	1
Scale Components.....	2
Display Symbols	3
Automatic Power-Off Function	3
Near Field Communication (NFC) Function	3
Installing (or Replacing) the Batteries	4
Measuring Your Weight on a Carpet	4

Operations

Setting the Unit	5
Measuring Your Weight Using the NFC Function.....	6

References

Maintaining the Scale.....	7
Before Requesting Service	7
Caution on Using the NFC Function	8
Specifications	8


Precautions

- Please read this booklet before using your scale.
- Place your scale on a solid level surface, such as tile, wood, or laminate flooring.
- Do not place your scale on a soft, wet or slippery floor.
If the scale must be used on a carpet, be sure to attach the plastic add-on feet as shown on Page 4.
- Your scale contains delicate, high-precision parts. Avoid extremes in temperature, humidity, direct sunlight, shock, moisture and dust.
- Do not step on your scale with your feet wet. Be sure that your feet are dry to prevent slipping or serious injury.
- Step on the center portion of your scale gently.
- A caregiver should attend when a disabled person uses the scale.
- Consult with your physician before you start a diet or exercises.
- Do not disassemble or modify the scale.


Indications for Use

- This scale is designed for personal use.


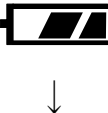


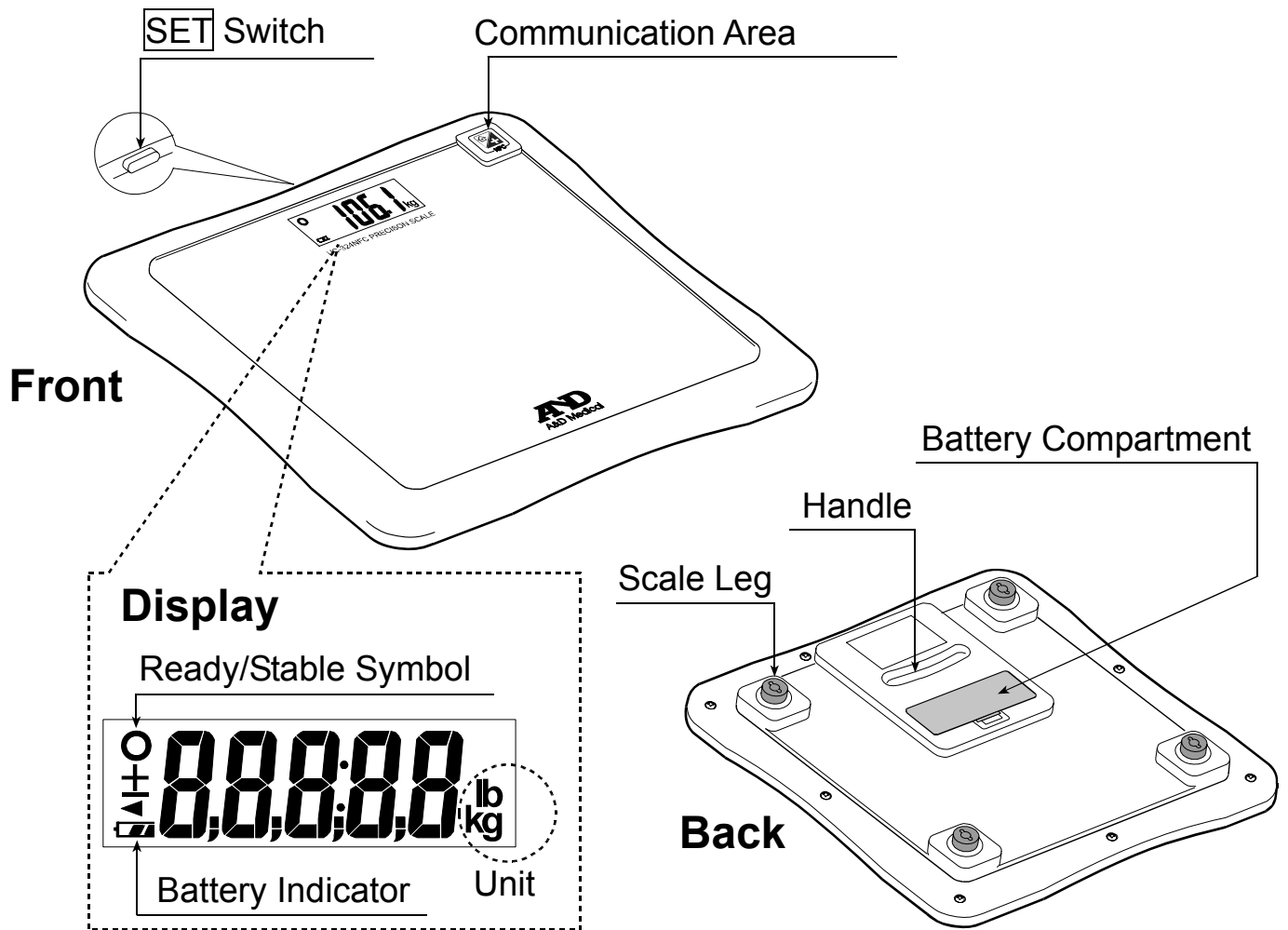
Scale Components

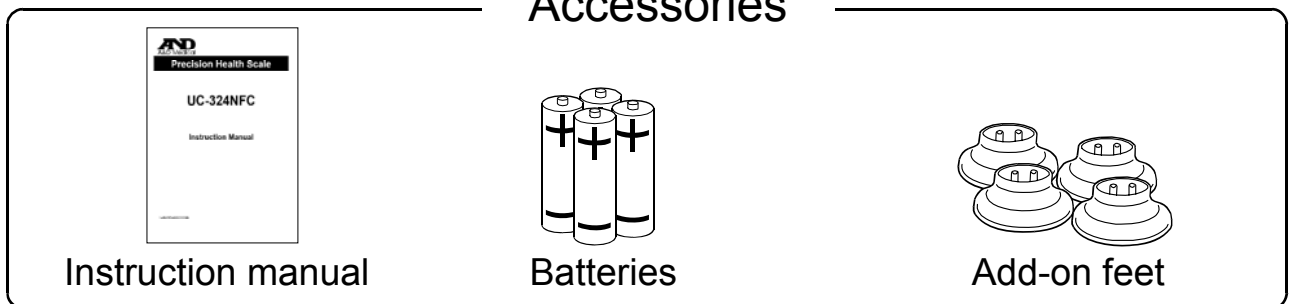

Accessories





Display Symbols

Symbols	Name	Description
	Ready/Stable symbol	This symbol is displayed when the measurement display is stable.
	Battery indicator	The normal battery power indicator with charge remaining
	Low battery indicator	If this symbol is displayed, replace all four batteries with new ones.
	WEEE label	_____
	CE mark	_____

Automatic Power-Off Function

The scale has an “Automatic power-off” function.

- If “0.00 kg /0.0 lb” and the Ready/ Stable symbol  are displayed for about forty-five seconds, the scale will automatically turn off.
- If the  symbol and the weight are displayed for about three minutes, the scale will automatically turn off.

Near Field Communication (NFC) Function

To transmit data to or from the scale, communication devices that comply with the communication specifications of the scale are required

Examples of communication devices that comply



- NFC-enabled cellular phone
- Personal computer with NFC reader/writer
- Access Point

Notes

- For information on communication methods and applications for the communication devices, refer to the instruction manuals for the devices.


Installing (or Replacing) the Batteries

Caution

- Insert each battery with its positive (+) and negative (-) terminals properly aligned with the corresponding symbols in the battery compartment.
- Improper installation of the batteries may cause battery leakage.
- Replace all four batteries with new ones, when the  symbol is displayed.
- Do not mix new and used batteries. It may cause damage to the scale or shorten the battery life.
- The  symbol does not appear when the batteries are completely drained.
- The battery life is dependant upon the environmental temperature, battery type and usage.
- Remove the batteries from the battery compartment, when the scale is not to be used for a long time. They may leak and damage the scale.
- The batteries provided with the scale are for trial only and may have a short life.

Steps for Installing (or Replacing) Batteries

1. Open the battery compartment cover on the underside of the scale.
2. Install four new batteries with the positive (+) and negative (-) terminals properly aligned with the corresponding symbols in the battery compartment.
3. Close the battery compartment cover.

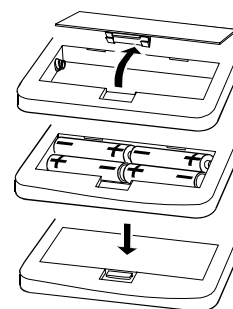


Measuring Your Weight on a Carpet

Place the scale on a solid level surface, such as tile, wood, or laminate flooring. If the scale must be used on a carpet, be sure to attach the plastic add-on feet.

Steps for Attaching the Add-on Feet

1. Each add-on foot has two pins. Match those pins to the holes on the scale leg and insert firmly.
2. Attach add-on feet on all four legs.

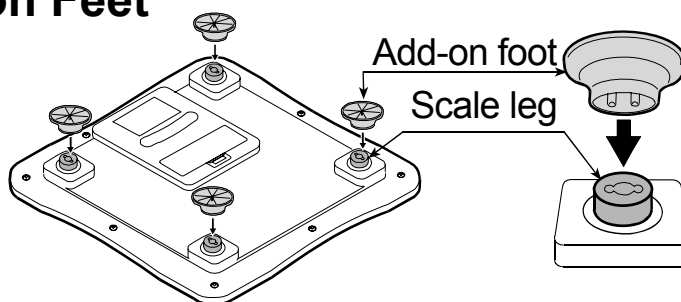


Notes

- Please be careful not to fall when using the add-on feet.
- We recommend using this scale on a solid and level surface.

Setting the Unit

Setting the unit manually

1. Confirm that the scale is turned off.
Press the **SET** switch. The scale beeps and displays the unit currently set.
2. Press the **SET** switch to select the unit.
3. After three seconds of non-operation, the scale confirms the selected unit with two beeps, displays “End” and automatically turns off.


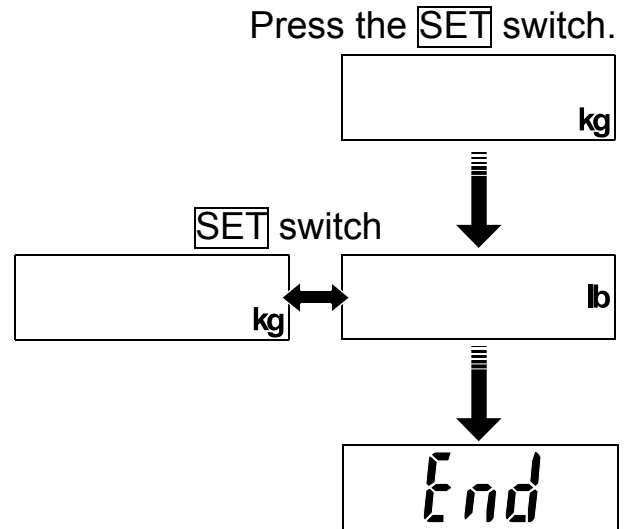


Setting the unit by the communication device

(Only for devices that rewriting of the unit information is enabled.)

1. Confirm that the scale is turned off.
Hold the communication symbol of the NFC-enabled device close to the communication area (NFC) of the scale.
2. When the data transmission is complete, the scale displays “End” and automatically turns off.

Example: Cellular phone

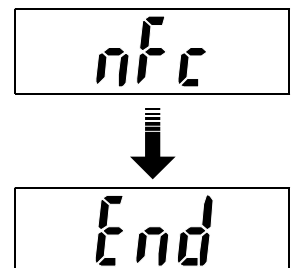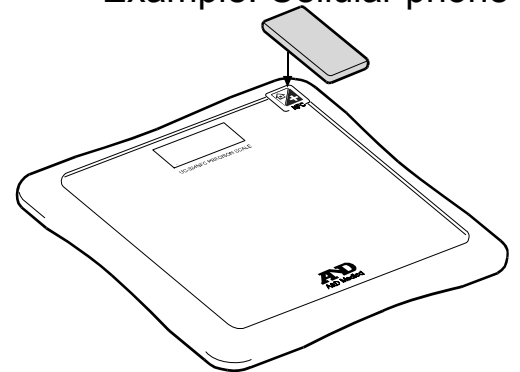

Note


- The NFC-enabled device may not be recognized when it is slanted or placed out of the communication area. If “E - 11” or “E - 12” appears, start from step 1 again.

Measuring Your Weight Using the NFC Function

After measurement, the communication device receives the data.


1. Confirm that the scale is turned off.
Step on the scale gently and stand still during the measurement. The scale turns on, displays all of the display segments.

2. Wait the finish of the measurement.
The process of the measurement is displayed with segment “-”.


3. Step off the scale when the stable symbol  is displayed with three beeps (Sound: pi pi pi).
If you do not step off the scale, the scale beeps every two seconds (Sound: pi) and the scale turns off after two minutes.

4. The scale displays all of the display segments.
Do not step on the scale.

5. The measurement result is displayed.

6. Point the communication symbol of the NFC-enabled device to the communication area () on the scale.

7. When the personal data transmission is complete, the scale displays “End”.

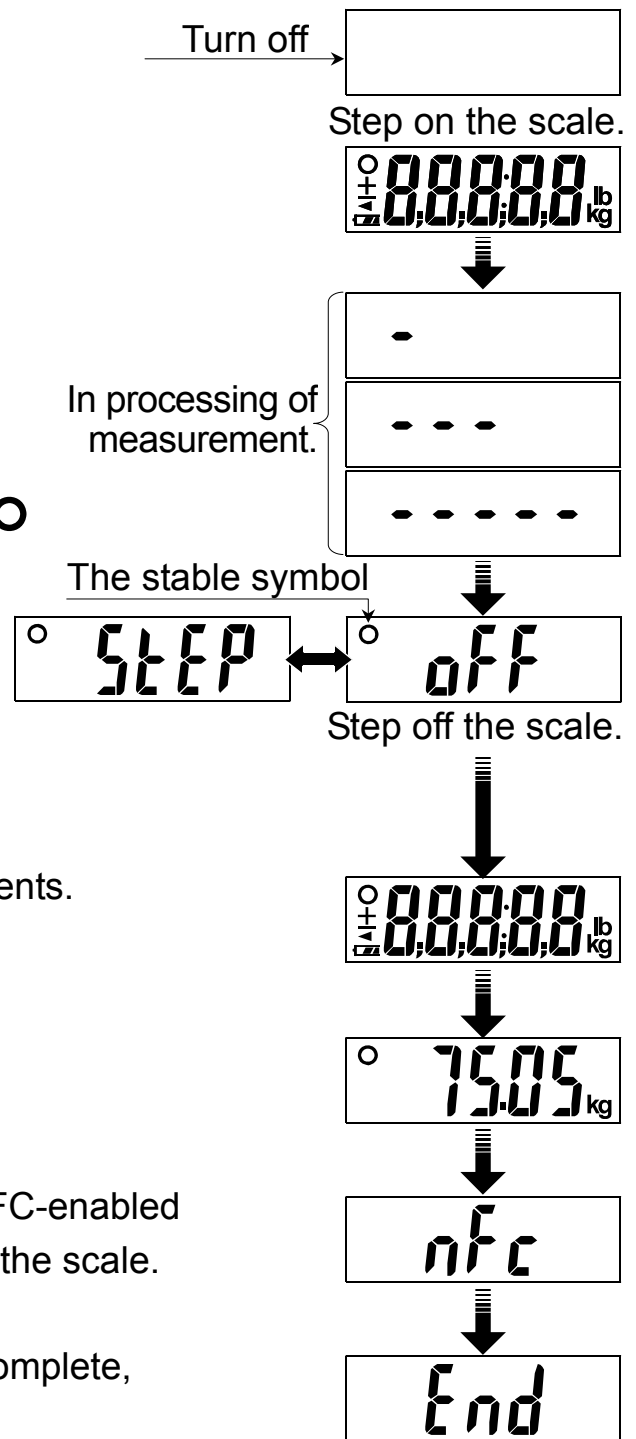


Notes

- The NFC-enabled device may not be recognized when it is slanted or placed out of the communication area. If “E - !!” or “E - !” appears, start from step 5 again.
- After the measurement result is displayed about three minutes, the scale will automatically turn off and the result will be deleted.

Maintaining the Scale

- Clean the scale with a soft dry cloth or a cloth moistened with water and a mild detergent. Never use alcohol, benzene, thinner or other harsh chemicals to clean the scale.
- Remove the batteries from the battery compartment, when the scale is not going to be used for a long time.

Before Requesting Service

Problems	Possible Reason / Corrective Action
E is displayed.	Exceeds the maximum capacity.
$E-0$ is displayed	
E_{rr} is displayed.	Exceeds the maximum capacity. Does not reach the minimum capacity.
$E-10$ is displayed.	The communication device may be slanted or placed out of the communication area of the scale.
$E-11$ is displayed.	
$E-12$ is displayed.	
H is displayed.	Exceeds the maximum capacity.
L is displayed.	
 is displayed.	Remove and reinstall the batteries. Or, replace all four batteries with new ones.
Nothing is displayed by tapping on the scale or pressing the SET switch.	
The measurement results are too much different from the expected values.	Place the scale on a solid level surface. Remove any foreign objects that might be touching the bottom of the scale. When the scale is placed on a carpet, be sure that the add-on feet are attached or try moving the scale to a level surface such as tile or wood.

If you can not locate and fix the problem, contact your supplier, or the A&D service group. The A&D service group will support authorized suppliers with technical information, spare parts and units.

Caution on Using the NFC Function


The scale has an NFC Forum Type 3 Tag wireless interface module built in. When the scale is used near wireless communication devices which use the same frequency as that of the scale (13.56 MHz) or use its harmonic frequencies, electromagnetic interference may occur. In that case, turn off the devices that are not in use, or place the devices 1 meter or more away from the scale.

Specifications

Type	UC-324NFC
Maximum capacity	150 kg / 330 lb
Minimum capacity	10 kg / 22 lb
Minimum display	0 to 100 kg: 0.05 kg, 100 to 150 kg: 0.1 kg 0 to 330 lb: 0.1 lb
Wireless communication	NFC wireless interface module
Setting items	Unit: kg or lb
Power source	Four type AA batteries (R6P or LR6, 1.5 volt x 4)
Battery life	Approximately 1 years (with two measurements a day using Alkaline batteries)
Dimensions	350 (D) X 350 (W) X 39 (H) mm
Mass	2.3 kg including the batteries
Accessories	Instruction manual, Batteries (4), Add-on feet (4)

Specifications are subject to change without prior notice or any obligation on the part of the manufacturer.

Compliance with European Directives

 The scale complies with the statutory EMC (Electromagnetic Compatibility) directive 2004/108/EC.

Note: Indoor use only. The displayed value may be adversely affected under extreme electromagnetic influences.



A&D Company, Limited

1-243 Asahi, Kitamoto-shi, Saitama 364-8585 JAPAN
Telephone: [81] (48) 593-1111 Fax: [81] (48) 593-1119

A&D INSTRUMENTS LTD.

Unit 24/26 Blacklands Way, Abingdon Business Park, Abingdon, Oxfordshire OX14 1DY United Kingdom
Telephone: [44] (1235) 550420 Fax: [44] (1235) 550485

A&D ENGINEERING, INC.

1756 Automation Parkway, San Jose, California 95131 U.S.A.
Telephone: [1] (408) 263-5333 Fax: [1] (408)263-0119

A&D Australasia Pty Ltd.

32 Dew Street, Thebarton, South Australia 5031 AUSTRALIA
Telephone: [61] (8) 8301-8100 Fax: [61] (8) 8352-7409

A&D KOREA Limited

한국에이.엔.디(주)
대한민국 서울시 영등포구 여의도동 36-2 맨하탄 B/D 8층
(8th Floor, Manhattan Bldg. 36-2 Yoido-dong, Youngdeungpo-ku, Seoul, KOREA)
전화: [82] (2) 780-4101 팩스: [82] (2) 782-4280